

HUMAC2015 EMG Driver Installation Instructions

1. Introduction.

- 1.1. The HUMAC includes drivers for the Delsys Trigno and Noraxon myoMOTION systems. This document describes how to install and register the HUMAC drivers.

2. Install the HUMAC2015 program.

- 2.1. The document *"300904 HUMAC2015 Software Installation Instructions"* contains complete instructions for installing the HUMAC2015 software.

3. Install the software from your EMG System manufacturer.


- 3.1. Delsys: Install the EMGworks program.
- 3.2. Noraxon: Install the MR3 and Acquire programs.

4. Installing the HUMAC/EMG Drivers.

- 4.1. Start the Command Prompt as an Administrator.

4.1.1. Windows 7.


- 4.1.1.1. From the **WINDOWS START** menu, select **START, ALL PROGRAMS, ACCESSORIES**.
- 4.1.1.2. Right-click **COMMAND PROMPT** and select **RUN AS ADMINISTRATOR**.

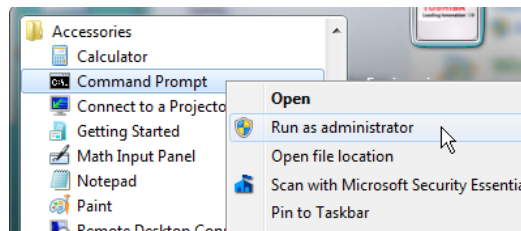


- 4.1.1.3. From the Windows *"The Following Program Will Make Changes..."* dialog box, click the **YES** button. Windows will open an **ADMINISTRATOR: COMMAND PROMPT** window.

4.2. Windows 10.

- 4.2.1. Type *"CMD"* in the Windows search box.
- 4.2.2. Right-click the **COMMAND PROMPT** and select **RUN AS ADMINISTRATOR**.

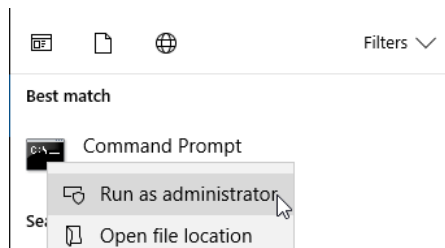

- 4.3. From the Windows “Do you want to allow this app to make changes...” dialog box, click the **YES** button. Windows will open an **ADMINISTRATOR: COMMAND PROMPT** window.
- 4.4. 64-bit Windows
 - 4.4.1. From the **COMMAND PROMPT**, enter (omitting quotes): “**CD \WINDOWS\SYSWOW64**” and press the **ENTER** key.
- 4.5. 32-bit Windows
 - 4.5.1. From the **COMMAND PROMPT**, enter (omitting quotes): “**CD \WINDOWS\SYSTEM32**” and press the **ENTER** key.
- 4.6. From the Command Prompt, type “**REGCSMIEMGDLL**” and press the **ENTER** key.
- 4.7. If Windows says you need to download “.NET Framework 3.5”, click **DOWNLOAD AND INSTALL THIS FEATURE**.
 - 4.7.1. After the installation completes, click the **CLOSE** button.
 - 4.7.2. Repeat the following: From the Command Prompt, type “**REGCSMIEMGDLL**” and press the **ENTER** key.
- 4.8. Windows should report “Types Registered Successfully”.

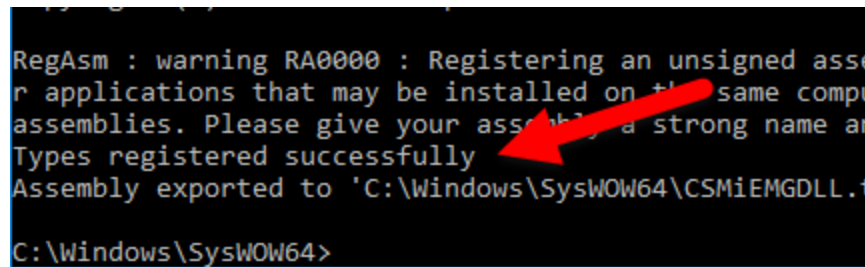

```
RegAsm : warning RA0000 : Registering an unsigned assembly for applications that may be installed on the same computer as signed assemblies. Please give your assembly a strong name and sign it with a private key.
Types registered successfully
Assembly exported to 'C:\Windows\SysWOW64\CSMiEMGDLL.t'
C:\Windows\SysWOW64>
```

- 4.9. From the Command Prompt, type “**EXIT**” and press the **ENTER** key to close the **COMMAND PROMPT** window.
- 4.10. The HUMAC EMG drivers are now correctly installed.